**Description**

IPS Empress® Direct is a light-curing, radiopaque composite, which is suitable for the fabrication of restorations that fulfill the highest demands. IPS Empress® Direct Effect is a flowable, light-curing, highly esthetic composite for the direct restorative therapy. IPS Empress Direct and IPS Empress Direct Effect cure with light in the wavelength range of 400-500 nm.

**Shades**

IPS Empress Direct is available in the following shades:

*Enamel shades*

A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, D2, D3, Bleach-L, Bleach-XL

*Dentin shades*

A1, A2, A3, A3.5, A4, IVA5, IVA6, B1, B2, C3, D2, Bleach-L, Bleach-XL

*Translucent shades*

Trans 20, Trans 30, Trans Opal\*

\*Not radiopaque due to its high transparency

IPS Empress Direct Effect is available in the following shades:

*Enamel shades*

Bleach XL

*Highly translucent Incisal shades*

Trans Opal, Trans 30

**Compositions****IPS Empress Direct**

The monomer matrix is composed of dimethacrylates (20-21.5 wt%, opalescent shade 17 wt%). The fillers contain barium glass, ytterbium trifluoride, mixed oxide, silicon dioxide and copolymer (77.5-79 wt%, opalescent shade 83 wt%). Additional contents: additives, initiators, stabilizers and pigments (<1.0 wt%). The total content of inorganic fillers is 75-79 wt% or 52-59 vol% (opalescent shade 60.5 wt% or 45 vol%). The particle size of the inorganic fillers is between 40 nm and 3 µm with a mean particle size of 550 nm.

**IPS Empress Direct Effect**

The monomer matrix is composed of dimethacrylates (42 wt% in the case of Effect Trans 30 and Effect Bleach XL, 33 wt% in the case of Effect Trans Opal). The fillers comprise barium glass, ytterbium trifluoride, highly dispersed silicon dioxide, mixed oxide and copolymer (58 wt% in the case of Effect Trans 30 and Effect Bleach XL, 67 wt% in the case of Effect Trans Opal). Additives, initiators, stabilizers and pigments are additional contents (< 1.0 wt%). The total content of inorganic fillers is 57 wt% or 36 vol% in the case of Effect Trans 30 and Effect Bleach XL, and 49wt% / 34 vol% in the case of Effect Trans Opal). The particle sizes of the inorganic fillers range between 40 nm and 3 µm. The mean particle size is 550 nm.

**Indications****IPS Empress Direct**

- Anterior restorations (Class III, IV)
- Restorations in the posterior region (Classes I and II)
- Tooth neck restorations (Class V, e.g. in the case of cervical caries, root erosion, wedge-shaped defects)
- Corrections of tooth positions and tooth shape (e.g. diastema closure, closure of interdental "black triangles", lengthening of the incisal edge)
- Direct veneers

**IPS Empress Direct Effect**

- As an intermediate or covering layer in the fabrication of esthetically demanding composite restorations
- Repair of composite restorations (in particular filling of voids, levelling out of porosities and minor chipping)
- As initial layer under Class I and II restorations
- Small restorations of all types
- Extended fissure sealing
- Splinting of mobile teeth
- Blocking out of undercuts
- Repair of composite and ceramic veneers

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## Contraindication

Placement of IPS Empress Direct and IPS Empress Direct Effect restorations is contraindicated

- if a dry working field cannot be established, or if the stipulated technique cannot be applied.
- if a patient is known to be allergic to any of the ingredients in IPS Empress Direct and IPS Empress Direct Effect.

## Side effects

In rare cases, components of IPS Empress Direct and IPS Empress Direct Effect may lead to sensitization. IPS Empress Direct or IPS Empress Direct Effect should not be used in such cases. To avoid possible irritation of the pulp, areas close to the pulp should be protected with a suitable pulp/dentin protector (selectively apply a calcium hydroxide-based preparation in areas close to the pulp and coat with a suitable liner).

## Interactions

Phenolic substances, such as materials containing eugenol/clove oil, inhibit the polymerization of methacrylate-based materials.

Consequently, the application of such materials together with IPS Empress Direct and IPS Empress Direct Effect must be avoided. Discoloration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

## Application

### 1. Shade determination

Clean the teeth prior to shade determination. The enamel shade is selected with the tooth still moist. The dentin shade can be taken either by placing the tab next to the prepared tooth or next to the cervical area of the canine.

### 2. Isolation

Adequate relative or absolute isolation using suitable auxiliaries, such as OptraGate® or OptraDam® Plus, is required.

### 3. Cavity preparation

Cavity preparation is carried out according to the requirements of the adhesive technique, i.e. protecting the tooth structure. Do not prepare any sharp internal edges nor additional undercuts in caries-free areas. The cavity geometry is mainly determined by the extend of caries and/or the old restoration. Enamel edges in the anterior region are bevelled. In the posterior region, only sharp enamel edges should be rounded (finishing diamonds, 25-40 µm). Subsequently, all residue in the cavity is removed with water spray. The cavity is dried with water- and oil-free air.

### 4. Pulp protection / Base

Do not apply a base material when using an enamel/dentin bonding agent. Only cover very deep areas close to the pulp with a calcium hydroxide material (e.g. ApexCal®) and subsequently use pressure-resistant cement (e.g. glass ionomer cement, such as Vivaglass® Liner). Do not cover other cavity walls, since they can be used to support the bond with an enamel/dentin adhesive.

### 5. Placement of a matrix / interdental wedge

Use a wrap-around matrix for cavities affecting the proximal area or a sectional matrix and wedge.

### 6. Conditioning / Application of the bonding agent

Condition the tooth structure and apply the bonding agent according to the Instructions for Use of the product in use. Ivoclar Vivadent recommends using the universal bonding agent Adhese® Universal, a single-component, light-cured adhesive suitable for direct and indirect restorations and all etching techniques.

### 7. Application of IPS Empress Direct and IPS Empress Direct Effect

- Since a large number of IPS Empress Direct Dentin and Enamel shades in different translucencies for the creation of anatomical details are available, any layering technique can be creatively used to achieve highly esthetic restorations. Apply IPS Empress Direct in layers of max. 2 mm (or 1.5 mm Dentin shades) thickness and, for an optimum result, adapt with e.g. OptraSculpt®/OptraSculpt® Pad. Apply IPS Empress Direct Effect in layers of max. 2 mm thickness.
- Sufficient exposure to the curing light prevents incomplete polymerization. Recommended curing times (exposure time) and light intensities are listed in Table 1. Hold the light emission window as closely as possible to the surface of the restorative material.
- When using a metal matrix, additionally polymerize the restoration from the oral and vestibular aspect after removing the matrix, if no Bluephase® curing light was used or if the light probe could not be optimally positioned (e.g. far away from the material or with divergent radiation).
- In many cases, a flowable composite is now used for the initial layer in order to create an even cavity floor and to simplify the adaptation of the subsequently used restorative material. The use of a flowable composite (e.g. Tetric EvoFlow®) as an initial thin layer is optional. This layer must be light-cured in a separate step (the instructions of the respective Instructions for Use must be observed).

### 8. Finishing / Checking the occlusion / Polishing

Remove excess material with suitable finishers or fine diamonds after polymerization. Remove proximal excess with diamond finishers, tungsten carbide finishers, finishing strips, or flexible finishing discs. Check the occlusion and articulation and apply appropriate grinding corrections to prevent premature contacts or undesired articulation paths on the surface of the restorations. Use silicone polishers (e.g. OptraPol®) as well as polishing discs and polishing strips to polish the restorations to a durable high gloss.

## Additional information

1. IPS Empress Direct can be combined with IPS Empress Direct Effect.
2. If necessary, additional IPS Empress Direct or IPS Empress Direct Effect material can be directly applied to the polymerized material. If the IPS Empress Direct or IPS Empress Direct Effect restoration has already been polished, it must first be roughened and wetted with Adhese® Universal before fresh IPS Empress Direct or IPS Empress Direct Effect is applied.

- IPS Empress Direct and IPS Empress Direct Effect should be at room temperature when they are applied. Cool temperatures render the material difficult to extrude.
-  For single use only. If IPS Empress Direct and IPS Empress Direct Effect are applied from the Cavifil or the application canula directly in the mouth of the patient, the Cavifil or the application canula must be discarded after use to prevent cross-contamination between patients.
- Syringes or Cavifils should not be disinfected with oxidizing disinfection agents.

### Warning

Unpolymerized IPS Empress Direct and IPS Empress Direct Effect should not come in contact with the skin, mucous membrane, or eyes. Unpolymerized IPS Empress Direct and IPS Empress Direct Effect may have a slight irritating effect and may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

### Storage

- Storage temperature 2-28 °C / 36-82 °F.
- Close syringes/Cavifils immediately after use. Exposure to light causes premature polymerization.
- Do not use IPS Empress Direct and IPS Empress Direct Effect after the indicated date of expiration.
- Shelf life: see information on Cavifils, syringes and packages.

**Keep material out of children's reach.**

**For use in dentistry only.**

The materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Table 1

| <b>Program</b> \ <b>Unit</b> | <b>Bluephase®<br/>C8</b> | <b>Bluephase®<br/>Style</b> | <b>Bluephase®</b> | <b>Bluephase®<br/>Style 20i</b> |
|------------------------------|--------------------------|-----------------------------|-------------------|---------------------------------|
| Turbo                        | –                        | –                           | –                 | 5 s                             |
| High Power                   | 15 s                     | 10 s                        | 10 s              | 10 s                            |
| Soft Start                   | –                        | –                           | 15 s              | –                               |
| <b>light intensity</b>       |                          | <b>exposure time</b>        |                   |                                 |
| ≥ 500 mW/cm <sup>2</sup>     |                          | 20 s                        |                   |                                 |
| ≥ 1000 mW/cm <sup>2</sup>    |                          | 10 s                        |                   |                                 |

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[www.ivoclarvivadent.us](http://www.ivoclarvivadent.us)

**Ivoclar Vivadent Shanghai Trading Co., Ltd.**

2/F Building 1, 881 Wuding Road, Jing An District

200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968

[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**Ivoclar Vivadent Marketing Ltd.**

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663

[www.ivoclarvivadent.co](http://www.ivoclarvivadent.co)

**Ivoclar Vivadent SAS**

B.P. 118, F-74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52

[www.ivoclarvivadent.fr](http://www.ivoclarvivadent.fr)

**Ivoclar Vivadent GmbH**

Dr. Adolf-Schneider-Str. 2

D-73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326

[www.ivoclarvivadent.de](http://www.ivoclarvivadent.de)

**Ivoclar Vivadent Marketing (India) Pvt. Ltd.**

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, Andheri (West), Mumbai, 400 053, India

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[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**Ivoclar Vivadent Ltd.**

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990

[www.ivoclarvivadent.co.nz](http://www.ivoclarvivadent.co.nz)

**Ivoclar Vivadent Polska Sp. z o.o.**

Al. Jana Pawla II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469

[www.ivoclarvivadent.pl](http://www.ivoclarvivadent.pl)

**Ivoclar Vivadent Marketing Ltd.**

Prospekt Andropova 18 korp. 6/

office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310

[www.ivoclarvivadent.ru](http://www.ivoclarvivadent.ru)

**Ivoclar Vivadent Marketing Ltd.**

Qlaya Main St., Siricon Building No.14, 2<sup>nd</sup> Floor

Office No. 204, P.O. Box 300146, Riyadh 11372

Saudi Arabia

Tel. +966 11 293 8345, Fax +966 11 293 8344

[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**Ivoclar Vivadent S.L.U.**

C/ Ribera del Loira nº 46, 5ª planta

28042 Madrid, Spain

Tel. +34 913 757 820, Fax +34 913 757 838

[www.ivoclarvivadent.es](http://www.ivoclarvivadent.es)

**Ivoclar Vivadent AB**

Dalvägen 14, S-169 56 Solna, Sweden

Tel. +46 8 514 939 30, Fax +46 8 514 939 40

[www.ivoclarvivadent.se](http://www.ivoclarvivadent.se)

**Ivoclar Vivadent Liaison Office**

: Tesvikiye Mahallesi , Sakayik Sokak

Nisantas' Plaza No:38/2, Kat:5 Daire:24

34021 Sisli – Istanbul, Turkey

Tel. +90 212 343 0802, Fax +90 212 343 0842

[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**Ivoclar Vivadent Limited**

Ground Floor Compass Building, Feldspar Close

Warrens Business Park, Enderby

Leicester LE19 4SE, United Kingdom

Tel. +44 116 284 7880, Fax +44 116 284 7881

[www.ivoclarvivadent.co.uk](http://www.ivoclarvivadent.co.uk)

**Ivoclar Vivadent, Inc.**

175 Pineview Drive, Amherst, N.Y. 14228, USA

Tel. +1 800 533 6825, Fax +1 716 691 2285

[www.ivoclarvivadent.us](http://www.ivoclarvivadent.us)