
Cervitec® Gel

Oral care gel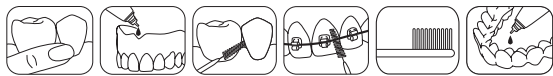

ivoclar
vivadent
clinical

English

Cervitec® Gel – The oral care gel

Ingredients

Aqua, Glycerin, Xylitol, Hydroxyethylcellulose, Panthenol, Laureth-23, Chlorhexidine digluconate, Sodium fluoride, Aroma, Sucralose.

Recommended application

- To clean, protect and maintain the healthy condition of the teeth, gums, peri-implant tissue and mucous membrane
- Providing hydration to the gingiva and the oral mucosa
- Oral care before and after the treatment of gingivitis, mucositis, denture stomatitis, periodontitis or peri-implantitis
- In case of high caries risk
- In case of restricted oral hygiene ability
- In case of bad breath (halitosis).

Depending on your needs, you may apply Cervitec Gel – directly on the gums, oral mucous membranes, on the tongue and teeth or the inner aspects of removable dentures.

- on the interdental brush to clean the space between teeth, orthodontic appliances or fixed restorations – for example bridges or implants.
- or use it to brush your teeth. Continue to brush once a day with your usual toothpaste.
- to your tray and wear it once or twice a day for 5 to 10 minutes. This procedure is recommended for a duration of 2 to 14 days every 3 to 4 months.

Duration

- Even when used for more than four weeks, no staining of teeth or problems with the sense of taste are to be expected.

Tip

Do not rinse after use! This supports the care effect of Cervitec Gel.

Note

- Contains sodium fluoride. Fluoride content: 900 ppm.
- For children younger than 6 years of age: Use only pea-sized amounts of gel. In order to prevent excessive swallowing, tooth brushing should be supervised. If fluoride is additionally administered, ask your physician or dentist.

Storage

- Store at 2–28 °C / 36–82 °F.

Keep out of the reach of children.

For use in dentistry only.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent AG