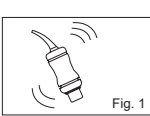
30001342 - ND3804
IFU FUJI PLUS GC CAPSULES EEP

GC Fuji PLUS™ CAPSULE

RADIOPAQUE REINFORCED GLASS IONOMER LUTING CEMENT IN CAPSULES

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Direct pulp-based inlays, onlays, crowns and bridges
- Restoration of resin inlays, onlays, crowns and bridges
- Cementation of all ceramic inlays
- Cementation of high strength (zirconia based) all ceramic crowns and bridges
- Cementation of metal, ceramic and fiber posts

CONTRAINDICATIONS

- Direct pulp capping
- Avoid use of this product in patients with known allergies to glass ionomer cement, methacrylate monomer or methacrylate polymer

COMPOSITION

Powder: Fluoro-alumino-silicate glass, initiator, pigment.
Liquid: Methacrylate, distilled water, polyacrylic acid, dimethacrylate, carboxylic acid, stabilizer.

DIRECTIONS FOR USE

Powder / Liquid ratio (g/g)	0,36 / 0,18
Mixing time (sec.)	10*
Working Time (23 °C / 73°F) (min., sec.)	200*
Setting Time(37 °C/99°F) (min., sec.)	415*

ISO 9917-2 Class 1, Luting Radio-opaque: Its radiopacity is equivalent to 2.1 mm or 210% aluminum. (The radiopacity of aluminum is comparable to dentin.)

1. TOOTH PREPARATION

- Prepare tooth in usual manner. For direct pulp capping, use calcium hydroxide.
- Clean the prepared teeth with pumice and water.
- Note: Apply GC Fuji PLUS CONDITIONER for 20 seconds using a cotton pellet or sponge.
- Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. DO NOT DESICCATE. Best results are obtained when prepared teeth are kept moist (glazing).

2. RESTORATION PREPARATION

- Make sure that the restoration is pretreated and handed according to manufacturer's instructions.
3. CAPSULE APPLICATION AND MIXING
 - Before activation, shake the capsule or put its side on a hard surface to loosen the powder (Fig. 1).
 - The capsule should be activated just before mixing and use immediately.

- Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
- The capsule should be activated just before mixing and used immediately.

- Immediately mix into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4,000 RPM) (Fig. 3).
- Note: When using a rotational mixing device, it is recommended not to exceed the centrifuge option.

4. DISPENSING

- Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- Make two clicks to prime the capsule (Fig. 4). To adjust the direction of the nozzle, hold the applicer with the capsule towards you and turn the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds (Fig. 2).
- Extrude directly into the restoration (Fig. 5).
- To remove the used capsule, push the applicer release button.

5. CEMENTATION

- Coat the internal surface of the restoration with sufficient cement and seat immediately. The working time is 2 minutes from start of mixing at 23°C (73°F). Higher temperatures will shorten working time.
- For more complex procedures requiring longer working time, it is recommended to refrigerate the capsules in the original aluminum foil.
- Start removing excess cement when the excess cement feels firm (Fig. 6).
- Finishing can be started 4 minutes 15 seconds after seating the restoration.

STORAGE

Recommended for optimal performance, store the capsules in the original aluminum foil in a cool and dark place 4-25°C (39-77.0°F).

SHADES

Yellow A3 (Vita® shade)
Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGE

50 capsules
Average contents per capsule: 0.36 g powder and 0.18 g (0.16 mL) liquid.
Minimum net volume of mixed cement per capsule: 0.19 mL.
Option: GC CAPSULE APPLIER (1 piece)

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCCA BUTTER can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- GC Fuji PLUS CONDITIONER contains 2% Ferric Chloride. It is not recommended to use GC Fuji PLUS CONDITIONER in combination with products containing Euphrasine as this may stain oral tissue.
- Do not use GC Fuji PLUS CAPSULE in combination with eugenol containing materials as eugenol may hinder GC Fuji PLUS CAPSULE from setting or bonding properly.
- This product is not indicated for filling or core build-up.
- In rare cases the product may cause sensitivity in some people. If any such changes are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gc.dental/europe>

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed/>) or contact us at Regulatory_gcc@gc.dental

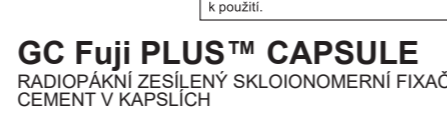
Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: <https://www.gcc.dental/europe>. In this way you will contribute to improve the safety of this product.

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