# MEMOREG® 2 / MEMOSIL® 2

DE	Gebrauchsanweisung.....	3	LV	Lietošanas instrukcija .....	62
GB	Instructions for use .....	7	LT	Naudojimo instrukcija .....	66
FR	Mode d'emploi .....	11	PL	Instrukcja obsługi .....	71
ES	Instrucciones de uso .....	16	HR	Upute za uporabu .....	76
IT	Istruzioni per l'uso .....	21	TR	Kullanma talimatı .....	81
PT	Instruções de uso .....	26	RU	Инструкция по применению	85
NL	Gebruiksaanwijzing .....	30	AE	تعليمات الاستخدام .....	90
SE	Bruksanvisning .....	35	KR	사용 설명서 .....	94
DK	Brugervejledning .....	39	SK	Návod na použitie .....	98
NO	Bruksanvisning .....	43	BG	Инструкции за употреба ..	103
FI	Käyttöohjeet .....	47	EE	Kasutusjuhend .....	108
GR	Οδηγίες χρήσης .....	52	SI	Navodila za uporabo .....	112
HU	Használati utasítás .....	57			

## Cartridge Delivery

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\* Gelson Luís Adabo, DDS, PhD, Elaine Zanarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD – Effect of disinfectant agents on dimensional stability of elastomeric impression materials. *J Prosthet Dent*, 81, 621-4, 1999

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Stand: 2023-04

**Medical Device – for use by dental health care professionals only.**

### **Intended purpose**

This dental accessory is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically, or they are accessories in the rehabilitation process.

### **MEMOREG® 2**

Addition-curing vinyl polysiloxane for direct application very high hardness

#### **Indications**

- Standard interocclusal records
- location material for intra-oral pin tracing template

### **MEMOSIL® 2**

Addition-curing vinyl polysiloxane for direct application transparent

#### **Indications**

- Front, occlusal record
- Transfer matrix for brackets (indirect bonding)
- Fixing of x-ray balls when planning the implants
- Matrix-walls (wall thickness max. 6 mm) for laboratory (pre-)pressing and injection technique of Signum veneering composites

## Contraindications

Memoreg 2 / Memosil 2 should not be used for conventional impressions since its high final hardness makes it very difficult to remove from the mouth. Undesired tooth extractions may result. The use of this product is contraindicated in case of known or suspected allergies against components of this product.

## Potential side effects

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

	<b>MEMOREG® 2</b>	<b>MEMOSIL® 2</b>
Working time* <sup>1</sup>	30 seconds	30 seconds
Minimum time in mouth* <sup>2</sup>	60 seconds	3 minutes
Shore A hardness	90	72
Product colours	pink	transparent

\*<sup>1</sup> Total working time (including mixing) at 23°C (73°F); shortened to 20s at oral temperature (35°C (95°F))

\*<sup>2</sup> Minimum time that the bite registration material must be in a closed mouth.

The data given are furnished to the best of our knowledge and are based on internal test results.

## Product application

Visually inspect for damage before first use. Damaged products must not be used. Mixing tips and bite registration tips are for single use only. To avoid cross contamination bite registration tips and mixing tips must be disposed of after initial use.

Dispense and discard a small amount of material from a 50ml cartridge before attaching a mixing tip. Attach mixing tip and fix by rotating clockwise. Dosing and mixing is done automatically on application. After use, leave the mixing tip on the cartridge to keep it closed. Immediately before the next use, remove the mixing tip and attach a new one, and again dispense and discard a small amount of material. When bite registration materials based on vinyl polysiloxane are touched with latex gloves, retraction solutions or freshly inserted composite fillings, it can change the curing behavior.

Memoreg 2/Memosil 2 should be used at room temperature (23°C (73°F)). Lower temperatures, for example when the materials are stored in a refrigerator can extend the working time, and higher temperatures can shorten it.

Hardened vinyl polysiloxanes are chemically resistant. Clothes can be permanently stained if the contact material is not completely cured.

Memoreg 2/Memosil 2 are applied with the green mixing tip (1:1). It can also be combined with the Bite Registration Tip.

### **General information**

Please pay attention to the following safety instructions and all processing instructions in other sections. This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself.

### **Disinfection**

Memoreg 2/Memosil 2 can be treated with an aqueous sodium hypochlorite solution (5.25% for up to 10 minutes\*). When using other disinfectant solutions suitable for silicones, follow the disinfection and processing instructions of the manufacturer.

\* Gelson Luís Adabo, DDS, PhD, Elaine Zananotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD – Effect of disinfectant agents on dimensional stability of elastomeric impression materials. *J Prosthet Dent*, 81, 621-4, 1999

## **Warnings and safety instructions**

Avoid contact with skin and eyes. May cause skin and eye irritation. Wear protective gloves according to EN 420, category III, with protection against chemical risks according to EN 374, type C. On contact with skin wash off with water and soap. If the material contacts your eyes, rinse with copious amounts of water for several minutes. If symptoms persist, consult an eye specialist. Do not swallow or ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. Do not leave any residual material in the sulcus or oral cavity, as this can lead to abscess formation. Surplus material on the bite fork or in the patient's mouth should be avoided as this may result in gagging, swallowing or aspiration. To further reduce these risks during impression taking, the patient should be positioned as upright as possible.

According to EU Medical Device Regulation, users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

## **Storage conditions**

Do not store above 25°C (77°F)! Store in a dry place. Do not use the material after the expiration date.

## **Disposal information**

Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

## **Complaint notice**

Please quote batch number and article number or UDI in all correspondence about the product. Article number **REF** and batch number **LOT** or UDI: See note on packaging.

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Kulzer GmbH
Kulzer, LLC