# VOCO

## Ufi Gel® hard C

**EN** Instructions for use  
**MD** EU Medical device

### Product description:

**Ufi Gel hard C** is a cold-curing permanently hard relining material for denture relinings. It is simple and quick to use for direct as well as indirect relinings. **Ufi Gel hard C** contains no methyl methacrylates. The set additionally contains an adhesive to achieve a permanent and stable bond.

### Indications:

- hard, permanent total or partial relinings to restore the functions of partial and complete dentures
- lengthening of denture margins

### Contraindications:

**Ufi Gel hard C** contains methacrylates, benzoyl peroxide, BHT and amines. Adhesive contains methacrylates. **Ufi Gel hard C** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

### Patient target group:

**Ufi Gel hard C** is suitable for use in all patients without any age or gender restrictions.

### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

### User:

**Ufi Gel hard C** should only be used by a professionally trained dental practitioner.

### Use:

#### 1. Preparation of the denture:

Check occlusion and carry out corrections, if necessary. Disinfect the prosthesis and dry thoroughly afterwards. Clean denture thoroughly with a brush and dry. Roughen all areas to be relined incl. buccal and labial margins with a suitable bur. Then clean and dry. Insulate all areas not to be relined (e.g. artificial teeth, labial and buccal areas beyond the roughened surfaces), e.g. with vaseline, to prevent unwanted contact with **Ufi Gel hard C**.

**Partial clasp dentures, anchors and attachments of dentures have to be blocked out with silicone (liquid) or wax with a low melting point.**

#### 2. Application of the adhesive

Exchange the transportation cap against the brush cap. Apply the adhesive to all surfaces to be relined and let it air-dry (30 s). The adhesive remains effective for about 10 min after application.

**Attention: Close the adhesive bottle tightly immediately after use and store in an upright position.**

**Important: For a secure bond use only Ufi Gel hard C adhesive.**

#### 3. Preparation of the cartridge

Put the **Ufi Gel hard C** cartridge into a **VOCO** dispenser (type 2) or any extruder gun with a suitable plunger.

Slight differences in the filling levels of the two cartridge chambers may occur for technical reasons.

Therefore, remove the transportation cap from the cartridge and extrude material until both chambers deliver uniformly. Then fix and lock the mixing tip with a 90° turn.

The attached mixing tip is intended for single use only. After use, store the cartridge tightly sealed. Check that material flows freely through the openings before reusing.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

#### 4. Application of Ufi Gel hard C

Apply **Ufi Gel hard C** evenly onto the prepared margin and/or base of the denture, avoid leaving excess material and remove any with a suitable instrument. Re-insert denture and have the patient exert slight pressure in occlusal position for 1 min and then carry out functional, chewing and swallowing movements for 2 min. Make sure that no material flows into the throat when relining an upper denture, especially at the transition from hard to soft palate (A-line).

#### 4.1 Partial or total dentures with undercuts

Remove denture 3.5 min after the beginning of application and remove excess material immediately with scissors or a scalpel. Re-insert into the mouth for a final occlusion, check and allow to cure for 2 min approx. Curing can also be completed in warm water, e.g. in a pressure pot at approx. 40 °C, if the relined denture is not to be re-inserted.

**Do not let material cure in contact with air** since oxygen will inhibit complete curing of the surface.

#### 4.2 Total dentures without undercuts

Excess material can be removed intra- or extraorally before final polymerisation.

Intraorally: remove excess material after 3 min (from beginning of application) with a suitable instrument. Let the denture cure for a further 2 min in the mouth until **Ufi Gel hard C** is completely cured. Extraorally: see position 4.1

#### 5. Finishing

The whole of the relined surface (also basal) is to be treated using standard tools for finishing (tungsten carbide bur, silicone polisher) and polishing (polishing paste, polishing disc).

beginning of application	0.5 min	apply
insert denture into the mouth	1 min	exert pressure
	2 min	functional movements
remove denture if necessary	1.5 min	remove excess
insert denture into the mouth	2 min	curing
remove denture	2.5 min	finishing, polishing
	9.5 min	

extraorally  intraorally  intra- or extraorally

### Warnings, precautionary measures:

- Store adhesive bottle tightly closed and in an upright position.
- Avoid contact of adhesive with skin. Wash contact points thoroughly with water and soap.
- In case of contact with the eyes rinse thoroughly with water and consult an ophthalmologist.
- The stated time periods refer to a room temperature of 25 °C and a temperature in the mouth of 35 °C. Polymerisation will be slower at lower temperatures, higher temperatures will accelerate polymerisation.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

### Storage:

Storage at 4°C-23°C. Do not use after the expiry date.

### Cleaning indications:

Dentures relined with **Ufi Gel hard C** can be cleaned with the usual cleansers and procedures. This applies to domestic as well as professional cleaning.

### Disposal:

Dispose of the product in accordance with local regulations.

### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Ufi Gel hard C** must be reported to VOCO GmbH and the responsible authority.

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